

Bone Funnels

- 1. Fill a clean sink with water and detergent in accordance with detergent manufacturer's instructions.
- 2. Brush, wipe, agitate, jet wash, or hand spray inside diameter of tube to remove excess bone grafting material
- 3. Remove the bone funnel from the sink and drain any excess detergent prior to placing in a second sink to rinse in clean water.
- 4. Rinse the bone funnel thoroughly with clean water or jet wash gun under the surface of the water.
- 5. Remove and drain the bone funnel before drying, using your preferred method.
- 6. Sterilize in autoclave between 134*C and 138*C (max 200*C) for the duration of the machine cycle. Cycles may vary according to the regulations and/or the law in force in the country involved.



www.atitan.com 877.284.8261